SINERGIA TEMPO MULTI

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The blanks are made of a multilayer PMMA in various shades (with non-toxic pigments) for a natural aesthetics without further processing.

The peculiarity of TEMPO MULTI consists in the layering from cervical to enamel; so the aesthetic result will vary according to the positioning of the structure into the disc.

INDICATIONS: CAD/CAM discs fit for fabrication of long-lasting temporary frameworks (more than 6 months) by dental milling systems. Please follow the instructions of the software and milling machines producers.

FRAMEWORK: Posterior: 2 bridge units and 1 attachment max.

Minimum thickness without veneering:

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FINISHING: Use carbide cutting discs (or other suitable tools) to separate the milled frameworks from the disc. The milled frameworks can be finished by tungsten carbide burs, pre-polished with silicone gummies and polished with regular polishing paste and soft brushes (goat hair).

IMPORTANT: Do not overheat acrylic when using burs, do not use alcohol based detergent for cleaning.

VENEERING: For veneering of the framework use suitable veneering composites and bondings. Thickness of framework should not be less than 0,4 mm after the process.

FIXATION: Use temporary fixation cement. Temporary cements containing Eugenol may decrease the adhesion effectiveness.

WARNINGS: By patients with possible individual allergic reactions, it is suggested to do first allergy testing to verify the product tolerance.

STORAGE: protect from direct sunlight and heat sources.

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