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Instructions for use

VITA VIONIC® BOND bonding system

Please read the instructions for use before using the product.

Intended use

VITA VIONIC BOND is a self-curing, two-component bonding system (BOND I and II) for permanently bonding VITA denture teeth from the VITA VIONIC product series, in the corresponding cavities of prosthetic bases made of compatible denture base materials.

Patient target group

No restrictions

Indication

For permanent bonding of VITA denture teeth from the VITA VIONIC product series in the corresponding cavities of prosthetic bases made of compatible denture base materials; see www.vita-zahnfabrik.com/vionibond

Contraindication

- For bonding prefabricated teeth that are not part of the VITA VIONIC product family.
- For bonding denture teeth whose outer contour does not match the contour of the cavity.
- For patients with allergies/sensitivities to MMA.

Product application/control

Please adhere to the following workflow when working with this product:

1. Check whether the prepared VITA VIONIC denture teeth fit the cavities in the prosthetic base
2. Mix VITA VIONIC BONDS I and II
3. Stir mixture for 30 sec.
4. Apply mixture to the adhesive surface of teeth and moisten cavities generously
5. Insert denture teeth and push them gently into place
6. Leave for 30 minutes
7. Cure using a pot (55°C, 2 bar, 20 min.) or by leaving at room temperature for min. 12 hours
8. Remove any excess by means of blasting (1 - 2 bar)

Detailed information on working with this product can be found in the document "Instructions for Use I Full Version" at www.vita-zahnfabrik.com/vionibond

Recommended product combinations

For bonding the digital VITA VIONIC VIGO prosthetic teeth or the millable VITA VIONIC FRAME prosthetic teeth to the definitive prosthetic base.

Storage/disposal

Store at a refrigerated temperature of 5 °C (41 °F) to 10 °C (50 °F). Store in a dry location. Protect from direct sunlight.

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems.

Technical data

| Physical properties | Value |
|---|------------------------|
| Bond strength with denture acrylic according to ISO 22112 | Complies with standard |

VITA VIONIC BOND I

| Chemical composition | Wt% |
|--|---------|
| Polymer based on methyl methacrylate CAS-No. 9011-14-7 | 70 - 80 |
| Co-polymer based on acrylates | 15 - 25 |
| methyl methacrylate CAS-No. 80-62-6 | < 5 |
| Triethylene glycol dimethacrylates CAS-No. 109-16-0 | < 4 |
| benzoyl peroxide CAS-No. 94-36-0 | < 2 |

VITA VIONIC BOND II

| Chemical composition | Wt% |
|--|------|
| methyl methacrylate CAS-No. 80-62-6 | > 95 |
| N,N-dimethyl-p-toluidine CAS-No. 99-97-8 | < 5 |

General risks/reporting incidents

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at www.vita-zahnfabrik.com/service_risks

Hazardous substances

VITA VIONIC BOND I + II

IMPORTANT!

| | |
|-----------|---|
| H225 | Highly flammable liquid and vapor. |
| H315 | Causes skin irritation. |
| H317 | May cause an allergic skin reaction. |
| H335 | May cause respiratory irritation. |
| P210 | Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition. No smoking. |
| P261 | Avoid inhalation of dust/smoke/gas/fog/vapor/aerosol. |
| P280 | Wear protective gloves/protective clothing/eye and face protection. |
| P235 | Keep at a cool temperature. |
| P403+P233 | Store well-sealed in an adequately ventilated location. Keep the container tightly closed. |

For VITA VIONIC BOND I only

P233 Keep the container tightly closed.

The safety data sheets can be downloaded at www.vita-zahnfabrik.com/vionibond or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not a required component of the product. Date of issue of this information: 04/20

After the publication of this information for use, any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com

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VITA Zahnfabrik has been certified, and the following products bear the mark:

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VITA VIONIC® BOND

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Instructions for use | short version

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VITA Zahnfabrik H. Rauter GmbH & Co.KG